



For a Wide Range of Use in Healthcare

MultiStrap is a new lifting accessory from Liko which completes the other lifting and repositioning aids in Liko's wide range. MultiStrap has been developed to meet a variety of needs in care and can be used to hold arms, legs and other parts of the body, for example, during treatment or washing.

Together with a patient lift, MultiStrap facilitates many heavy lifting tasks such as wound care (e.g. burn injuries), hygiene procedures, treatment (e.g. catheterizing or dressing wounds), examinations, placement of bedpans and dressing/undressing. MultiStrap can also be used to turn a patient to the side.

MultiStrap can be used individually or in pairs. For turning in bed, we recommend using it in pairs with one of the wider models.

Different models and areas of use

MultiStrap Standard

Width 10 cm (4 inches)

MultiStrap Standard is usually the easiest to apply and allows better access to a larger area of skin. It is especially suitable for lifting and holding an arm or leg.

MultiStrap Wide

Width 25 cm (10 inches)

MultiStrap Wide has a greater support area and distributes pressure better due to its reinforcement. It can be used to lift a patient's leg or behind or to turn a patient in bed.

MultiStrap Ultra

Width 40 cm (16 inches)

Developed especially for bariatric patients but can also be used to turn lighter patients in bed. Has reinforcement to distribute pressure more evenly.

A FixStrap 135 cm is included with all MultiStraps and is used to fix one end of the MultiStrap to the bed when turning a patient.



Repositioning in bed can be carried out quickly and effectively using MultiStrap, for example to facilitate hygiene procedures.

PRODUCT INFORMATION

Product designation	Product no.	Maximum load	Fabric	Width
MultiStrap Standard, incl. FixStrap	3695115	200 kg (440 lbs)	polyester	10 cm (4 in.)
MultiStrap Wide, incl. FixStrap	3695116	200 kg (440 lbs)	polyester	25 cm (10 in.)
MultiStrap Ultra, incl. FixStrap	3695117	200 kg (440 lbs)	polyester	40 cm (10 in.)

Accessories	Product no.
FixStrap 135 cm	3696001
Spatula 600 IC	3684002
(can facilitate placement of MultiStrap Standard and Wide)	

Read the relevant instruction guide for further information on areas of use and practical application. You can also contact your nearest Liko representative for information and advice. You will find our addresses at www.liko.com.



MultiStrap is a class 1 medical device product.

Design and Quality by Liko in Sweden

Liko is quality-certified in accordance with ISO 9001 and its equivalent for the medical device industry, ISO 13485.

Liko is also certified in accordance with environmental standard ISO 14001.



www.liko.com

Manufactured by:

Liko AB
Nedre vägen 100
975 92 Luleå
Sweden
info@liko.se

Local representative in UK

Hill-Rom Limited
Clinitron House, Ashby Park
Ashby de la Zouch,
Leicester LE65 1JG
Tel: 01530 411000
Fax: 01530 411555
info@liko.co.uk

To find your local Liko representative,

please visit www.liko.com or call
+46 (0)920 47 47 00



A **Hill-Rom** Company